FALKE

ERGONOMIC
SPORT SYSTEM

FALKE Achilles

de	deutsch	4
en	english	8
fr	français	12
da	dansk	18
es	español	22
it	italiano	26
nl	nederlands	30
no	norsk	34
sv	svenska	38

de

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en

These instructions for use are also available to download at
https://www.falke.com/assets/download/29/MS_Achilles_Gebrauchsanleitung-1929.pdf

Dear customer,

Thank you for putting your trust in this high-quality FALKE product. The FALKE Achilles is a product with high medical and quality standards.

Please read these instructions for use carefully and keep them in a safe place. If you have any questions, please contact your attending physician, your specialist retailer or our Customer Service team. The instructions for use are intended to ensure the safe use of the product.

Intended use

The FALKE Achilles is a sock which has been especially developed to address Achilles tendon complaints. The gentle compression means that the raised nubs inside the sock exert pressure on the tissue on the left- and right-hand side of the Achilles tendon. This provides a massage effect during physical activity or when playing sports, which in turn helps to stimulate the metabolism, improve circulation and initiate the necessary regeneration processes. The massage effect promotes the reduction of any adhesions on the tendon inside the tendon sheath.

The regeneration of tissue can also help to reduce the pain in the area of the Achilles tendon.

Indications

- Achillodynia (tendinitis or tendinopathy, paratendinopathy)
- Later rehabilitation stage following an Achilles tendon injury

Usage risks

Important information

- No modifications may be made to the product. Failing to observe the instructions for use may impair the performance of the product.
- If you notice any unusual changes (e.g. if symptoms become worse), please consult your physician immediately and do not continue using the product.
- Consult your attending physician before combining the FALKE Achilles with any other products (e.g. compression socks).
- The massaging effect of FALKE Achilles is activated by movement during sports or during a normal day and that's when it reveals its full potential. The product should not be worn during sport for more than 2-3 hours per day; otherwise side effects (e.g. skin irritation) may occur. It is possible, however, to wear the sock for longer during your normal day.
- It is important to walk or run when wearing the product.
- No side effects that affect the entire body are currently known. This assumes that the FALKE Achilles is being put on and used correctly.
- Remove the sock immediately if you notice any allergic reactions or if the raised silicone nubs are causing skin irritations. A slight redness of the skin due to the raised silicone nubs is not a cause for concern, as it indicates improved circulation.
- Only ever wear the FALKE Achilles with ankle-high trainers. Higher shoes such as hiking boots or ski boots can significantly increase the pressure exerted by the raised silicone nubs, which can cause pain.

Contraindications

No pathological hypersensitivities are currently known. The FALKE Achilles should only be worn following consultation with your doctor if you have any of the following medical conditions:

- Skin disease or injury in the area of application, in particular if there are indications of an inflammation (excessive heat, swelling or redness)
- Acute Achilles tendon inflammation (tendinitis)
- Sensory disorders or circulatory problems in the legs or feet (e.g. due to diabetes)
- Lymph drainage disorders with oedema formation in the lower extremities
- Undefined swellings in the area of application
- Undefined pain in the area of application
- Damaged skin or skin disorders; in particular skin infection in the area of application
- Allergic reactions of the skin in the area of application

Usage notes

- Please observe the instructions on how to put on the sock.
- Please make sure that the raised silicone nubs are positioned on the sides of the Achilles tendon.
- Please note the “Right” and “Left” markings on the sock.
- Correct the position of the sock if necessary.

Cleaning instructions

Please observe the washing and care instructions in order to guarantee the product's mode of action over a long-term basis.

Information on re-use

The product is intended for the treatment of a single patient.

Indications for serious incidents:

If your health is seriously impaired in connection with wearing the product, please contact the FALKE company or its representative and the competent authority for medical devices in your country without hesitation. You can find the corresponding FALKE contact addresses on page XY.

Warranty

The legal provisions of the country where the product was purchased apply.

Country-specific warranty regulations between the retailer and the purchaser apply where relevant.

If you wish to lodge a warranty claim, please initially contact the place where you purchased the product.

Please follow our usage and care instructions. These instructions are based on our own experience, and help make sure that our medically effective products remain fully functional for a long period of time. You will only be able to enjoy the benefits of the products if they are working perfectly.

Failing to observe the instructions can limit the warranty.

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BA Sportprodukter AB
FALKE Nederland B.V.